

The Sentinel Max Level 2 Isolation Gown

These full-coverage, single-use, non-sterile, non-surgical isolation gowns are intended to protect patients and healthcare personnel from the transfer of microorganisms, bodily fluids, and particulate material. Two size options are available to ensure a proper fit. All gowns are 100% sourced and manufactured in the United States of America.


ADG in collaboration with Maddox Defense is proud to offer this 100% sourced and made in the USA isolation gown.



Integrated elastic cuffs



Gown SMS material attached to gown for tie closures

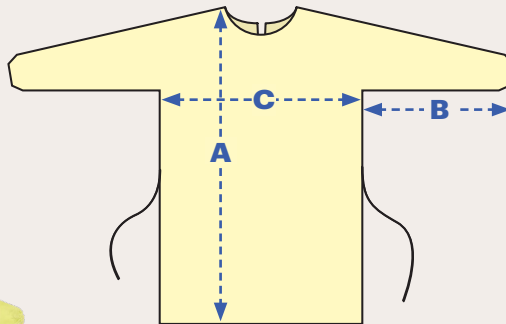

FRONT



BACK

	LARGE	XLARGE
Part Number	ABDL2-99L-US	ABDL2-99XL-US
A Length	47"	48"
B Sleeve Length	23"	23"
C Chest	27"	30"

Specifications

MATERIALS

Medium-weight Tri-Layer SMS fabric

COLOR

Yellow

DESIGN

Full coverage gown with secure neck and waist ties using gown SMS material attached to the gown, integrated elastic cuffs

FABRIC WEIGHT

29 GSM

USABILITY

Single Use (Disposable)

STERILIZATION

Non Sterile

DECLARATIONS

Not made with natural rubber latex

CERTIFICATIONS

ANSI/AAMI PB70 – Barrier Level 2

STORAGE CONDITIONS

Store at Room Temperature
Dry Environment
Remain in Original Packaging

SHELF LIFE

10 years



To purchase this gown, contact ADG at info@adg.com.



2550 Auburn Court
Auburn Hills, MI 48326

info@adg.com
www.adg.com


Continuous development may cause the information and specifications contained herein to change without notice. ADG reserves the right to change, modify, or discontinue any products. Images may be slightly different than actual products.




User Instructions

The Sentinel Max Level 2 Isolation Gown


Donning

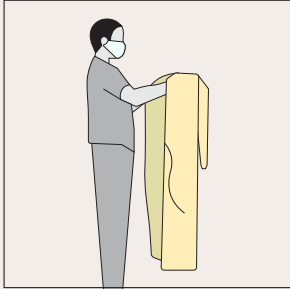


1. Begin by opening the gown. Hold the gown away from the body and allow it to unfold completely.

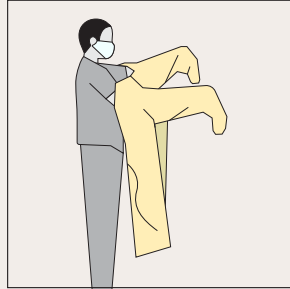

2. Slip both arms into the sleeve and hands through the elastic cuff. Adjust the front of the gown to fully cover the torso and expose the full back of the gown. Pull out wrinkles and make sure the gown is smooth on your back.

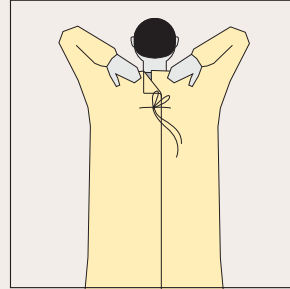


3. Reach behind the neck to grab the ties. To ensure full coverage, make sure the layers are overlapping without a gap. Firmly tie the back of the gown together.

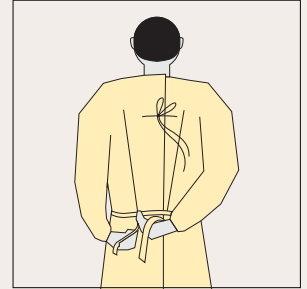

4. Grasp the rear waist ties and securely tie the gown together while ensuring there is overlap of the gown.


Doffing

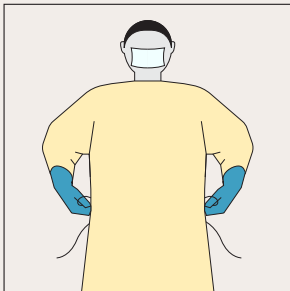


1. To remove the gown, insert your thumbs between the ties and the gown and pull forward to break the ties, touching the outside of the gown only with gloved hands.



2. With both hands grasp the gown in the front from the waist away from your body.

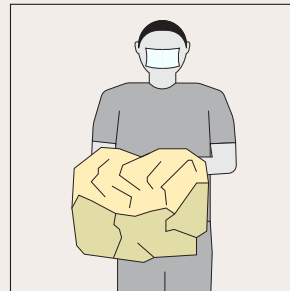

3. While removing the gown, fold or roll the gown inside-out into a bundle.

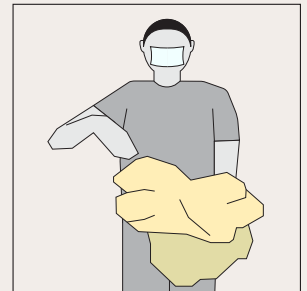

4. As you are removing the gown, peel off your gloves at the same time, only touching the inside of the gloves and gown with your bare hands. Place the gown and gloves into a waste container.



If the end user has questions concerning the safe operation of this product, please consult the local supervisor or safety program administrator.



User Instructions

The Sentinel Max Level 2 Isolation Gown

Donning

1. Begin by opening the gown. Hold the gown away from the body and allow it to unfold completely.
 2. Slip both arms into the sleeve and hands through the elastic cuff. Adjust the front of the gown to fully cover the torso and expose the full back of the gown.
 3. Reach behind the neck to grab the tie. Grab the tie by reaching behind the neck toward the upper opposite right shoulder. Firmly tying together the back of the gown.
 4. Grasp the front tie, wrap it around to the back of the gown and tie securely.
-

Doffing

1. To remove the gown, insert your thumbs between the ties and the gown and pull forward to break the ties, touching the outside of the gown only with gloved hands.
2. With both hands grasp the gown in the front from the waist away from your body.
3. While removing the gown, fold or roll the gown inside-out into a bundle.
4. As you are removing the gown, peel off your gloves at the same time, only touching the inside of the gloves and gown with your bare hands. Place the gown and gloves into a waste container.



If the end user has questions concerning the safe operation of this product, please consult the local supervisor or safety program administrator.

